



ADAPTER 4I22

SQUARE ADAPTER

SQUARE ADAPTER

stainless steel









Square 4-hole fixed female adapter. They are used exclusively for the manufacture of lower extremity prostheses.

Made of high resistance stainless steel. Indicated for patients with a maximum weight of 100 kg.

Fixed pyramidal receiving adapter, used in combination with 3I15, 3A40 and 3I20 knee joints. Compatible with any pyramid receiving adapter, for modular leg prostheses.

These combinations provide angle and translation modifications in the sagittal and anterior plane, as well as regulation of internal and external rotation.



ACTIVITY LEVEL	   			
PATIENT WEIGHT LIMIT				
IMPACT LEVEL	1-4	RESISTANCE	100 KG	
SKU	4I22	UNIT WEIGHT	80 GR	

CAUTION!

Do not expose the prosthetic components to environments that favor the corrosion of metal, as well as the degradation of the elements that make up the adapter. If it is exposed to any of these environments and the adapter has traces of elements that favors its deterioration, the warranty offered by O and P will be lost.

WARRANTY

The 4I22 adapter warranty only applies if it is used under the conditions described in this sheet. This product is insured for 1 year using exclusively O and P modular products.

SQUARE ADAPTER


INSTRUCTIONS FOR USE

Fixed square adapter with pyramid receiver.

The union with the opposite piece in the modular system is through the M8 x 14 mm opressors.

Corrections can be made at any time during alignment, try-in and after completion of the prosthesis. To exchange any part, disassemble it. To keep the previous alignment, unscrew only the two adjoining opressors.

For the definitive use, secure it with a torque wrench set to 13 NM. The application of Loctite 242 threadlocker is necessary.¹

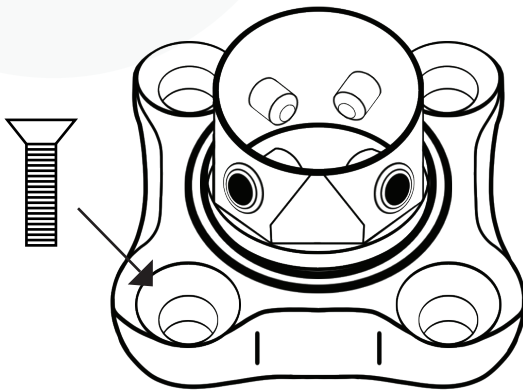

ACTIVITY LEVEL				
PATIENT WEIGHT LIMIT	80 KG	100 KG	125 KG	150 KG
IMPACT LEVEL	1-4	RESISTANCE	100 KG	
SKU	4122	UNIT WEIGHT	80 GR	